

Installation Guide

For adults and children (above 2 years)

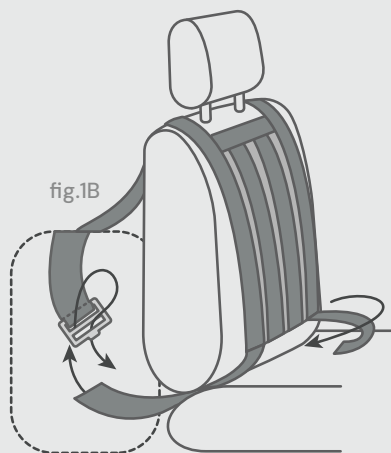


fig.1A

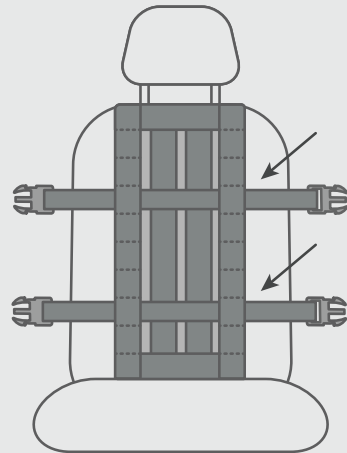


fig.2

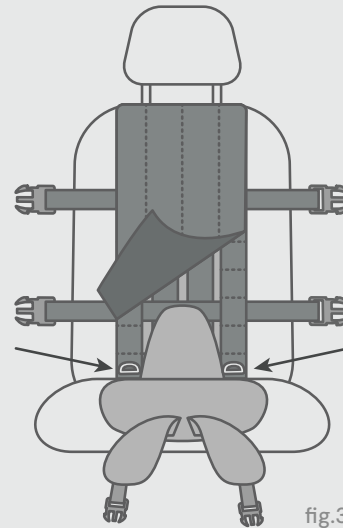


fig.3

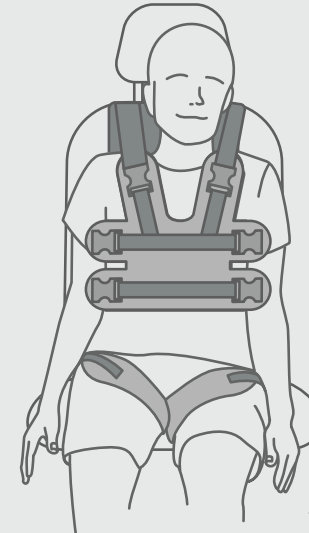


fig.4



fig.5

Simple steps to setup atubi for your vehicle

- Push the bottom straps of the backrest through the gap between the car seat and backrest, then fasten the buckles and tighten until the system fits firmly. (fig. 1A, fig. 1B)
- Adjust the height of the transverse restraining belts if necessary. (fig. 2)
- Fasten the leg harness if required. For a child, press the Velcro flap of the leg harness against the Velcro on the backrest. For adults, use the two D-strings attached to the bottom of the backrest for extra support. Cover all straps with the soft backseat cover. (fig. 3)
- Secure the occupant with the chest harness and leg harness. (fig. 4)
- Fasten the vehicle safety belt — this remains compulsory whenever the restraint is used. (fig. 5)

Important Safety Disclaimer:

This device has been designed and constructed to the best of our ability for disabled patients, that need additional support in a car. Nevertheless, we regret that the management, its employees and agents, must stipulate that they are absolutely unable to accept liability or responsibility for injury or damage of any nature whatsoever whether arising from negligence or any other cause howsoever which is suffered by any person who purchases the device and/or uses it.